
Spinny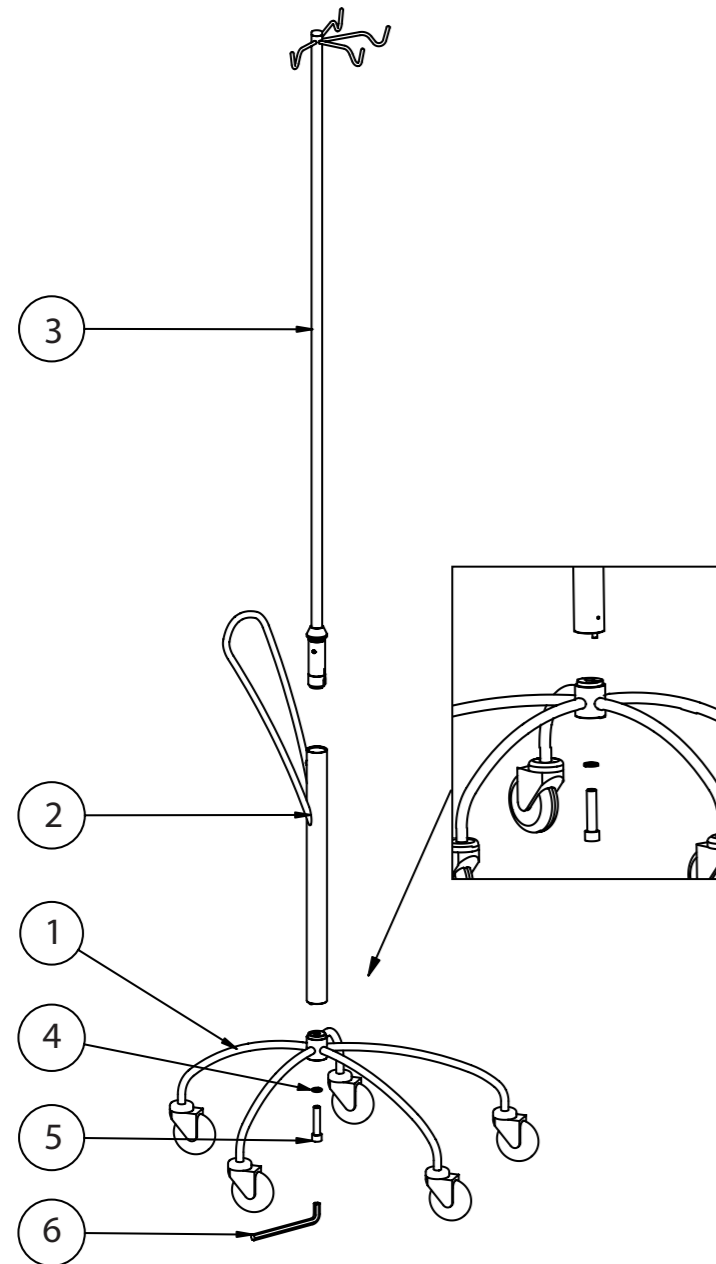

Handleiding (NL)

Manuel (FR)

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Manual (EN)

created by NIDES

R5.00

NL

Toepassing

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Application

Spinny is a drip stand/IV pole intended to assist with the administration of enteral and parenteral liquids and to suspend medical appliances such as IV pumps.

Liquid bags and bottles can be hung from the hooks and medical appliances can be attached to the upper section of the pole. The wheel base is for the patient to move around with the drip stand. Lines and tubing can be hung over the bar of the handle.

Use

The upper part (pole with hooks) can be separated from the wheeled base (pole with wheels). This way, the upper part can be moved together with the patient to a wheelchair, bed or stretcher when necessary. The upper part can be attached to the bar of the wheelchair, bed or stretcher. Move the upper part with care, since the attached bags and/or pumps can make it top-heavy. When putting the upper part back in place, make sure that the black coupling is fully connected to the wheeled base.

In order to make certain that the upper part is always in the right position, the rod of the upper part is shaped so that there is only one way to place the pole in the wheeled base. The hooks are located opposite the handle. This provides the best balance when bags or bottles of liquid are hanging from the hooks.

Safety measures / warnings

- Do not use the drip stand for other purposes than the one described above and not as a toy for children.
- Do not stand on the legs of the wheeled base even just to reach the hooks of the drip stand.
- Do not hang pumps, drip bags or any other heavy objects on the handle, the drip stand could tip over.
- When the upper part is attached to a wheelchair the hooks are at head/eye level of bystanders. Users and bystanders must be warned of the potential danger.
- Move the upper part carefully when bags and/or pumps are attached it can be top-heavy.
- Should the drip stand become defective call technical services as soon as possible for any necessary repairs.
- Maximum load to hooks: 4 x1000ml infusion bags/ bottles, or equivalent. Weight must be evenly divided on hooks. Maximum 1 bag/bottle per hook.
- Maximum load to pole: 4x 2,5kg of medical equipment, or equivalent. Weight must be placed as low as possible in the same direction as the hooks.

- incidents that directly or indirectly led, might have led or might lead to death, temporary or permanent health deterioration of the user or any other person or a public health risk) that has occurred related to this medical device, should be reported to the manufacturer and / or distributor and to the competent authority in the Member State in which the user is located.

Assembly

Before assembly of the drip stand make sure that all of the required parts are present:

- 1) wheeled base
- 2) lower pole with handle
- 3) upper part
- 4) locking washer
- 5) allen screw
- 6) allen wrench

A. Fix the lower pole (2) to the wheeled base (1) with the allen screw (5). Do not forget to place the locking washer (4) between the screw head and the wheeled base! Make sure that the small peg of the lower pole fits into the hole of the wheeled base.

B. Then place the upper part (3) in the lower pole (2). Make sure that the black coupling of the upper part fully connects to the lower pole.

Recommendations for maintenance

Clean the drip stand regularly with a mild detergent. There is no need to disassemble the stand to clean it.

When cleaning, pay extra attention to the fastening points of the wheels, the legs and the handle. If necessary, remove any dirt built up between wheels and wheel cap which can interfere with manoeuvrability.

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For more information, please contact:

Indes Production Management
Colosseum 20
7521 PT Enschede
The Netherlands
Tel: +31 (0)53 480 39 20
Fax: +31 (0)53 480 39 22
info@indes.eu
www.indes.eu